

BoNee®

Bladder Injection Needle

Purposefully Designed for Delivery

The BoNee® needle is used to deliver injectable materials into the urinary bladder wall during transurethral endoscopic procedures.

5 Fr body

Flexible body for rigid and flexible cystoscopes

4 mm tip

Designed with urologists to facilitate bladder wall injection

Tip with protection cap

Cap avoids damaging the working channel of the cystoscope during needle insertion

Pointer stop

reduces bladder perforation risk

Locking connector

Tuohy Borst connector allows physician to lock needle to cystoscope's working channel. It facilitates puncture and needle rotation

Short tip

reduces injection pain and bleeding

Coloplast Delivers Confidence

It all started in 1954 with Elise Sorensen, a nurse who wanted to help her sister regain confidence and control. After having an ostomy operation, her sister was afraid to go out in public for fear of leakage. Elise worked with engineer Aage Louis-Hansen to develop the world's first adhesive ostomy bag for greater protection and control. Today, Coloplast is an environmentally conscious global leader in ostomy care, wound and skin care, and urology care products that improve lives for millions of people around the world.

To Order Call Toll Free 800.258.3476

This product may be ordered directly from Coloplast.

Bonee® Bladder Injection Needle	Length (cm)	Purchase UOM	Qty per UOM	Item
Rigid Cystoscope Needle	35	EA	1	NB1035
Flexible or Rigid Cystoscope Needle	70	EA	1	NB1070

BONEE® - NEEDLE FOR BLADDER INJECTIONS BRIEF STATEMENT

Indication

The Bonee needle is used to deliver injectable materials, amongst other Botulinum toxin, into the urinary bladder wall during the transurethral endoscopic procedure.

Contraindications

The Bonee needle is contraindicated for use in patients with the following conditions:

- coagulation disorders
- any known allergies to the medical device materials
- any contraindication related to cystoscopy or to the injected substance

Warnings

Refer to the instructions for use of the injected substance and the clinical injection protocol.

Precautions

The choice of the size of the needle is the responsibility of the physician.

This type of needle must only be used by trained and experienced professionals.

Reuse of this single use product may create a potential risk to the user. Reprocessing, cleaning, disinfection and sterilization may compromise product characteristics

which in turn create an additional risk of physical harm to or infection of the patient.

Potential Complications

Potential complications related to injection into the bladder wall include: bleeding, bladder wall perforation. Potential complications related to the passage of the cystoscope-needle unit into the bladder include: tears or perforation of the meatus and/or the urethra if the device is advanced with the needle protruding, pain, urinary tract infection. For injected substance related adverse effects, please refer to its instructions for use.

The information provided is not comprehensive with regard to product risks. For a comprehensive listing of indications, contraindications, warnings, precautions, and adverse events refer to the product's Instructions for Use. Alternatively, you may contact a Coloplast representative at 1-800-258-3476 and/or visit the company website at www.coloplast.com.

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.

Minneapolis, MN
12/15/2021 PM-04815

References
1. Data on file at Coloplast.