



Testicular replacement

A patient guide

Inside this guide

Testicle overview	4
What causes a missing testicle?	5
Things to consider when facing testicular loss	6
Torosa [®] testicular implant	8
Insurance information	10

Losing one or both of your testicles isn't an easy topic to discuss, but it can be a reality for many.

Regardless of age, the loss of a testicle not only has impacts on physical appearance but can also impact your confidence and your self-esteem.¹ The fact is, there is an alternative that can help restore part of what was lost.



Restore a more natural appearance.

There is a solution.


Review this guide to learn more and then talk with your doctor about your treatment options.

Testicle overview

The testicles—also called testes—are two oval-shaped organs in the male reproductive system responsible for producing sperm and male hormones such as testosterone.

The testicles are contained in a sack of skin called the scrotum which hangs outside the body behind the penis.

Disorders or conditions affecting the testes can lead to complications, including hormonal imbalances and infertility.^{2,3} Additionally, the loss of the testes can have a negative psychological effect and may impact your self-esteem and/or self-confidence.¹

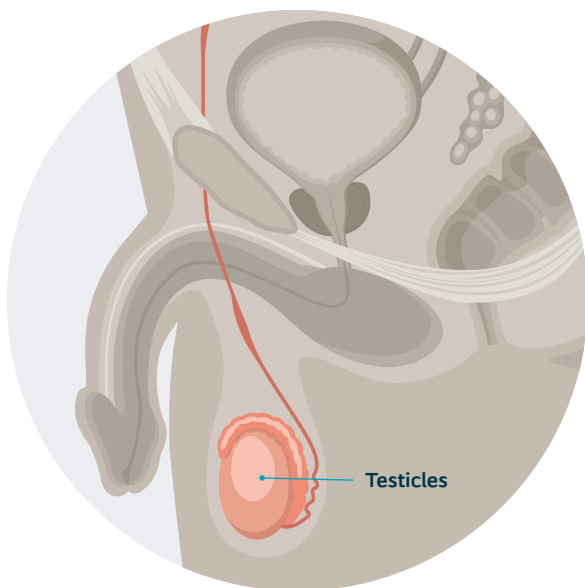



The absence of a testicle has been shown to be a psychologically traumatic experience for males of all ages.⁴

Some of the causes of a missing testicle include the following:⁴

- Testicular cancer
- Testicular trauma
- Testicular torsion
- Undescended testicles
- Born without a testicle

Things to consider when facing testicular loss


If you have had or expect to have one or both of your testicles removed (typically done through a procedure called an orchiectomy), there are many things to consider.

You may want to talk to your doctor about:

- Impact on testosterone level
- Fertility and family planning
- Testicular implants²

Testicular implants are more than just a cosmetic solution –

50-60%
of men reported
improved body image
after receiving
an implant.⁵

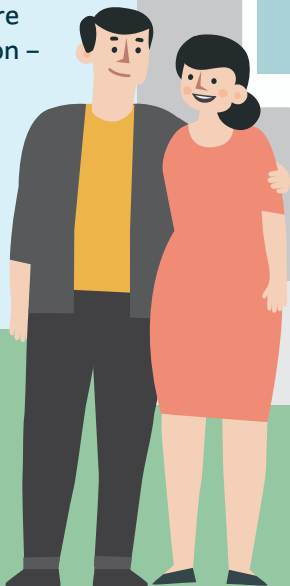

Testicular implants have been shown to reduce the psychological impact resulting from loss or absence of a testicle.⁴

Testicular implants

Following testicle removal, you may be worried about the cosmetic effect of missing a testicle, but there is hope. A testicular implant can help you maintain the look, feel, and shape of your scrotum.⁶

If you are interested in testicular implants, it is important to discuss timing and preparation for receiving a testicular implant after testicle removal. Depending on patient and doctor preference, the implant can be placed at the same time as the testicle is removed or it can be done later in a separate procedure. There may be ways you'll need to prepare if you have two separate surgeries.⁵ Discuss this further with your physician.



To listen to a story of someone who has faced testicle removal and has experienced life with testicular implants, [scan QR code](#)


www.coloplastmenshealth.com/testicular-implants/



Torosa[®] Testicular Implant

The Torosa implant is a saline-filled testicular implant that is placed completely inside the body, typically during an outpatient procedure. The Torosa implant aids in creating and restoring a more natural look and feel of the testicles in the scrotum.⁶

The Torosa implant has been rigorously tested and is the only testicular implant with FDA approval.

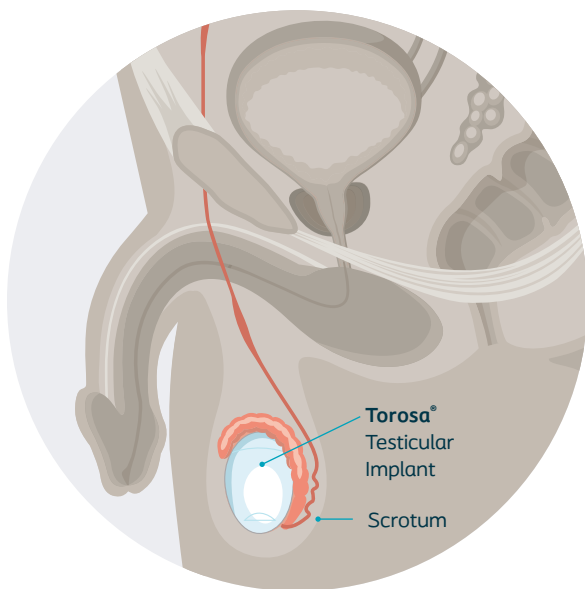

88% of men
implanted with a testicular implant
reported that they would undergo the
surgery again if it was necessary.¹



Actual size: Extra Small

Talk to your doctor about the Torosa implant

98% of patients believe it's important to be offered a testicular implant but only **47%** of them are.^{7,8} It's not too early to ask if a testicular implant is right for you—and share your preferences around your treatment plan, including whether the Torosa implant can be placed at the same time as the testicle removal.

Torosa implants are available in multiple sizes to help match the size of the testicle(s) that were removed. Your physician will put together a specific plan for your recovery process. Refer to the Important Safety Information at the end of this brochure to view potential complications.

93% satisfaction
with the Torosa implant⁶

94%
would recommend to a friend¹



Small



Medium



Large



Insurance coverage

Testicular implants, including the Torosa implant, are prescribed by a physician. Cost and insurance coverage can vary based on many factors.

To determine if testicular implants are included in your insurance policy, **call 855-230-7611** to speak to one of our insurance specialists or:

Visit this page for more information about payment and insurance options

coloplast.to/testicular-implant-insurance



[Scan QR code](#)

The staff at your physician's office can also help you determine coverage.



Next steps

1. **Talk to your doctor about the Torosa implant.**
2. **If you are not currently receiving care from a urologist who specializes in testicular implants, visit our Physician Finder to search for one near you.**

www.coloplastmenshealth.com/find-a-physician/



[Scan QR code](#)

3. **Visit our website to view patient stories and learn more.**

www.coloplastmenshealth.com



[Scan QR code](#)

Torosa® Saline-Filled Testicular Protheses

Important Safety Information

Torosa Saline-Filled Prosthesis is a surgically implanted artificial testicle designed to replicate the size, shape, and feel of one or two testicles following surgical removal, or the absence of a testicle (agenesis) in males.

Indications

The Coloplast Torosa Saline-Filled Testicular Prosthesis is intended for use when cosmetic testicular replacement is indicated i.e., in the case of agenesis or following the surgical removal of a testicle.

Contraindications

The implantation of testicular prosthetic implants is contraindicated in the presence of infection or abnormal growth (neoplasm).

Warnings

A testicular implant in patients with pre-existing enlargement of the scrotum (varicocele) may result in persistent pain. Due to the nature of silicone implants, testicular implants should not be considered lifetime implants and require replacement surgery over time. Torosa should typically not be implanted in patients with a documented sensitivity to silicone. These patients should discuss the risks of this implant with their physician. Patients with disorders such as lupus, scleroderma and myasthenia gravis should discuss the risks of this implant with their physician. Sepsis, hemorrhage or thrombosis may result from the placement of any foreign object in the body. An oversized implant can lead to potential complications such damage or loss of tissue (necrosis) or foreign body reaction leading to formation of blood clots (thrombosis). Excessive scarring and/or tightening around the implant (capsular formation or contracture) may occur.

Precautions

Discuss all available treatment options with your doctor to understand the risks and benefits of a testicular implant.

Potential Complications

The following device-related and procedure-related events were experienced during the clinical study for this device: discomfort/pain, swelling (edema), extrusion, displacement/migration, hematoma, abnormal tissue formation (keloid, fibrosis, granuloma), implant deflation, fluid accumulation (inguinal area), constipation, numbness/weakness (neuropathy) in legs/feet, and surgical site infection (abscess).

Advice to Patient

Resize of the implant may be elected in the future for young males implanted prior to body maturation.

This treatment is prescribed by your physician. Discuss the treatment options with your physician to understand the risks and benefits of the various options to determine if a testicular implant is right for you.

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.

Coloplast was founded on passion, ambition, and commitment. We were born from a nurse's wish to help her sister and the skills of an engineer.

Guided by empathy, our mission is to make life easier for people with intimate healthcare needs. Over decades, we have helped millions of people live more independent lives and continue to do so through innovative products and services.

Globally, our business areas include Ostomy Care, Continence Care, Wound and Skin Care, Interventional Urology and Voice and Respiratory Care.

For more information please visit
www.ColoplastMensHealth.com

References:

1. Clifford TG, Burg ML, Hu B, Loh-Doyle J, Huguen CM, Cai J, Djaladat H, Wayne K, Daneshmand S. Satisfaction With Testicular Prosthesis After Radical Orchiectomy. *Urology*. 2018 Apr;114:128-132.
2. Testicular Cancer. Urology Care Foundation. <https://www.urologyhealth.org/urology-a-z/t/testicular-cancer>. Accessed February 2024.
3. Testicular Torsion. Urology Care Foundation. <https://www.urologyhealth.org/urologic-conditions/testicular-torsion>. Accessed July 2024.
4. Bodiwala D, Summerton DJ, Terry TR. Testicular prostheses: development and modern usage. *Ann R Coll Surg Engl*. 2007 May;89(4):349-53.
5. Hayon S, Michael J, Coward RM. The modern testicular prosthesis: patient selection and counseling, surgical technique, and outcomes. *Asian J Androl*. 2020 Jan-Feb;22(1):64-69.
6. Data on File at Coloplast.
7. Dieckmann KP, Anheuser P, Schmidt S, Soyka-Hundt B, Pichlmeier U, Schriefer P, Matthies C, Hartmann M, Ruf CG. Testicular prostheses in patients with testicular cancer - acceptance rate and patient satisfaction. *BMC Urol*. 2015 Mar 13;15:16.
8. Robinson R, Tait CD, Clarke NW, Ramani VA. Is it safe to insert a testicular prosthesis at the time of radical orchidectomy for testis cancer: an audit of 904 men undergoing radical orchidectomy. *BJU Int*. 2016 Feb;117(2):249-52.

Coloplast Corp. Minneapolis, MN 55411
Interventional Urology Surgical Support 1-800-258-3476